GENEZIS

OWNERS MANUAL WELCOME TO THE WORLD OF COLOURS!

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 DO NOT OPERATE THIS WHEELCHAIR WITHOUT FIRST READING AND UNDERSTANDING THIS OWNERS MANUAL. IF YOU ARE UNABLE TO UNDERSTAND THE WARNINGS, CAUTIONS AND INSTRUCTIONS, CONTACT YOUR COLOURS DEALER. YOU IGNORE THIS WARNING, YOU MAY FALL, TIPOVER OR LOSE CONTROL OF THE WHEELCHAIR AND SERIOUSLY INJURE YOURSELF OR OTHERS OR DAMAGE THE WHEELCHAIR.

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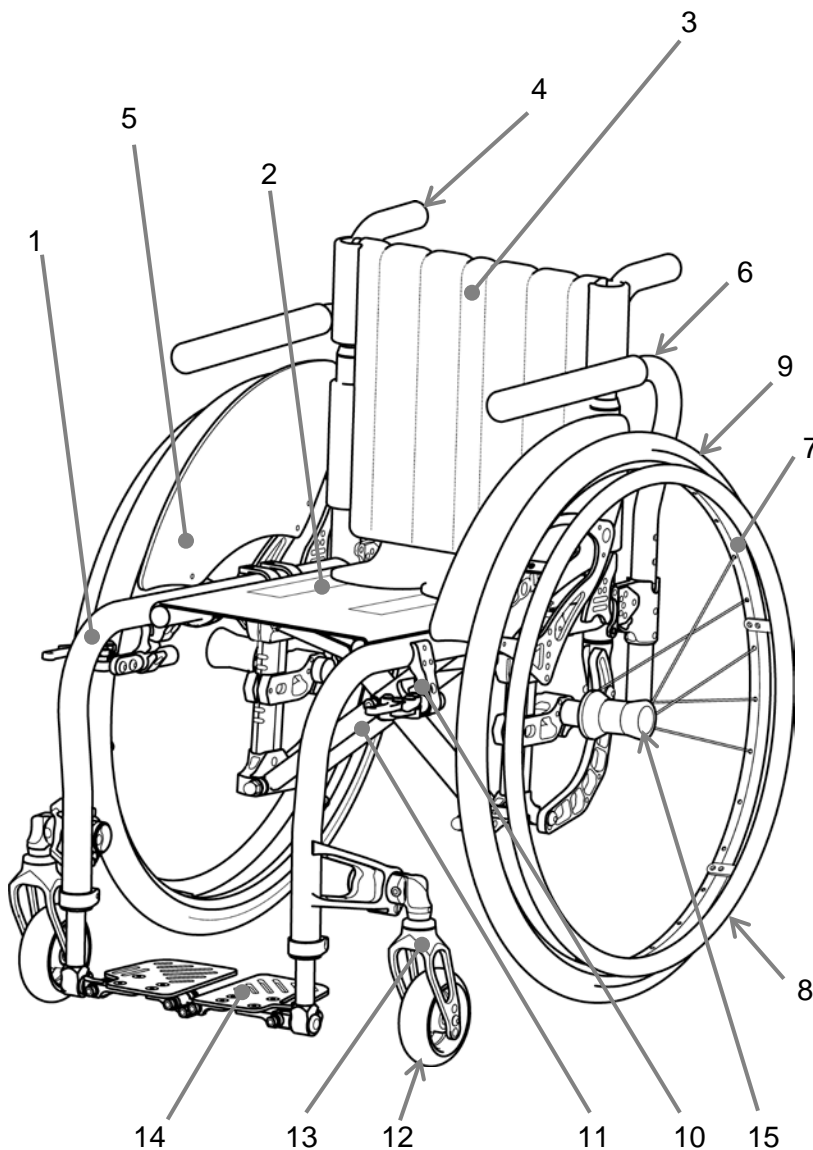
4. GENERAL WARNINGS-1

A. ⚠️WEIGHTLIMIT

NEVER exceed the weight limit of 265 pounds (120 kg) for combined weight of rider and items carried.

If you do exceed the limit, damage to your chair, a fall, tip-over or loss of control may occur and cause severe injury to the rider or others.

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B. ⚠️GETTINGTOKNOWYOURCHAIR

Every wheelchair is different. Take the time to learn the feel of this chair before you begin riding. Start slowly, with easy, smooth strokes. If you are used to a different chair, you may use too much force and tip over. If you use too much force, damage to your chair, a fall, tip-over or loss of control may occur and cause severe injury to the rider or others.

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C. ⚠️POWERDRIVE

If you install a power drive add-on system to your chair and your chair does not have a power adaptable frame, you may damage your chair and you will void your warranty.

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4. GENERAL WARNINGS-2

D. ⚠️ TO REDUCE THE RISK OF AN ACCIDENT

1. BEFORE riding, you should be trained in the safe use of this chair by your health care advisor.
2. Practice bending, reaching and transfers until you know the limit of your ability. Have someone help you until you know what can cause a fall or tip-over and how to avoid doing so.
3. Be aware that you must develop your own methods for safe use best suited to your level of function and ability.
4. NEVER try a new maneuver on your own until you are sure you can do it safely.
5. Get to know the areas where you plan to use your chair. Look for hazards and learn how to avoid them.
6. Use anti-tip tubes unless you are a skilled rider of this chair and are sure you are not at risk to tip over.

If you fail to heed these warnings damage to your chair, a fall, tip-over or loss of control may occur and cause severe injury to the rider or others.

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When pushing downward, do not place your hand below the seat bars, such as to grasp the seat frame bar. Your fingers may get caught, leading to injury.

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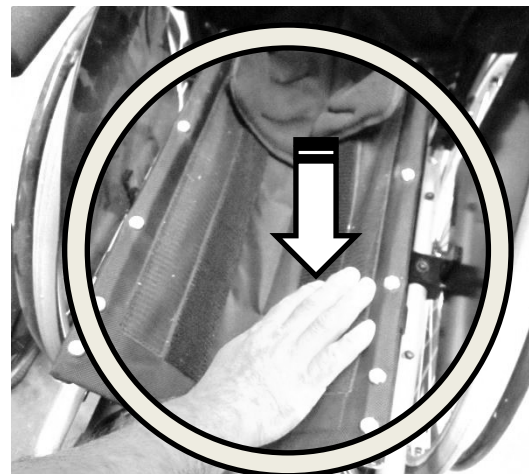
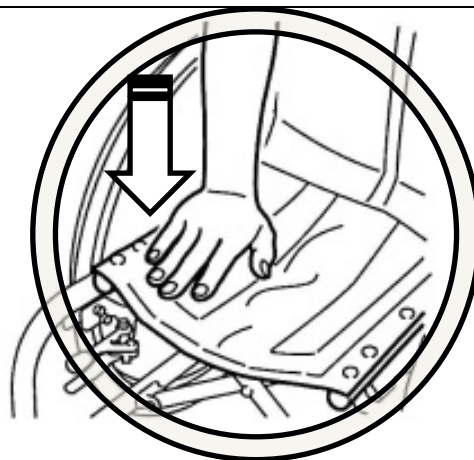
Grasp the center of the front and back sections of the seat and lift it up.

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
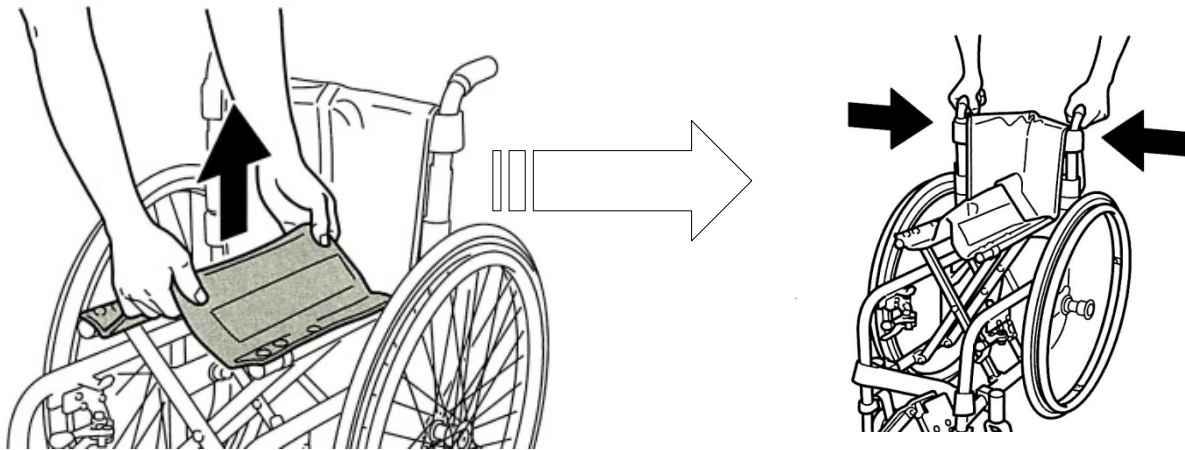
If folding is difficult, first fold up the the footplates and then fold the chair.

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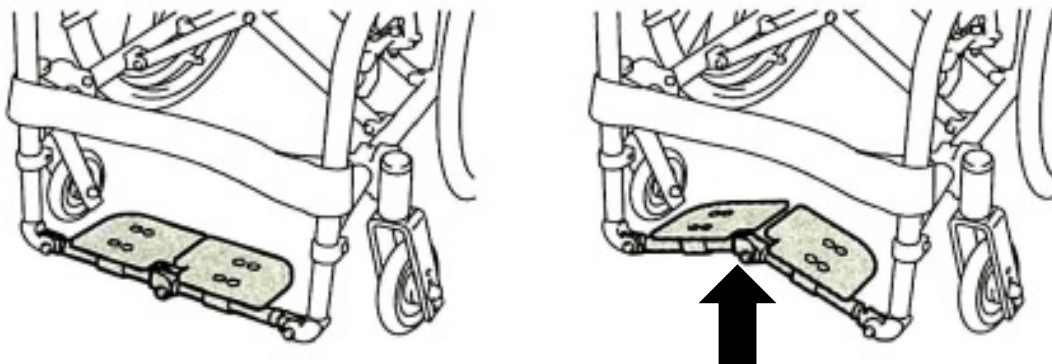
5.SET-UP & ADJUSTMENT - 1

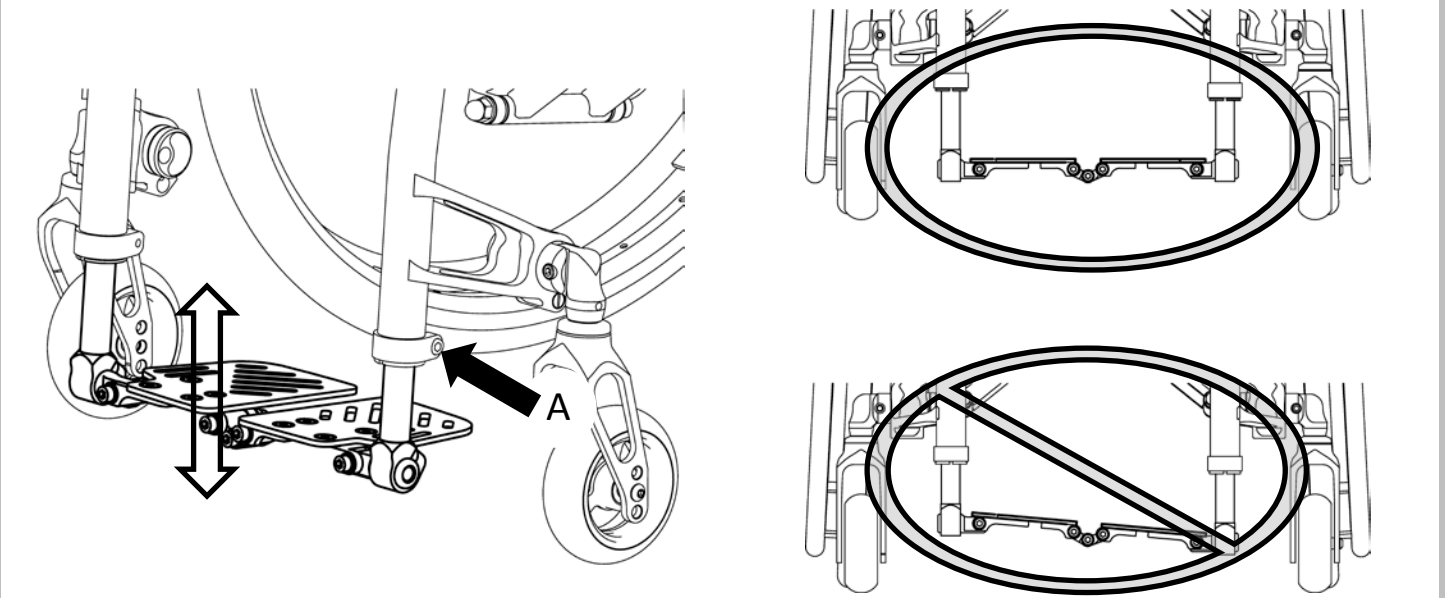Footrest Height Adjustment

1. Loosen set screw (A) using 5mm hex key.
2. Slide footrest tube up or down inside frame tube to desired height.
3. Tighten set screw.

For optimum performance, the foot plate should always be parallel to the floor (horizontal to the ground).

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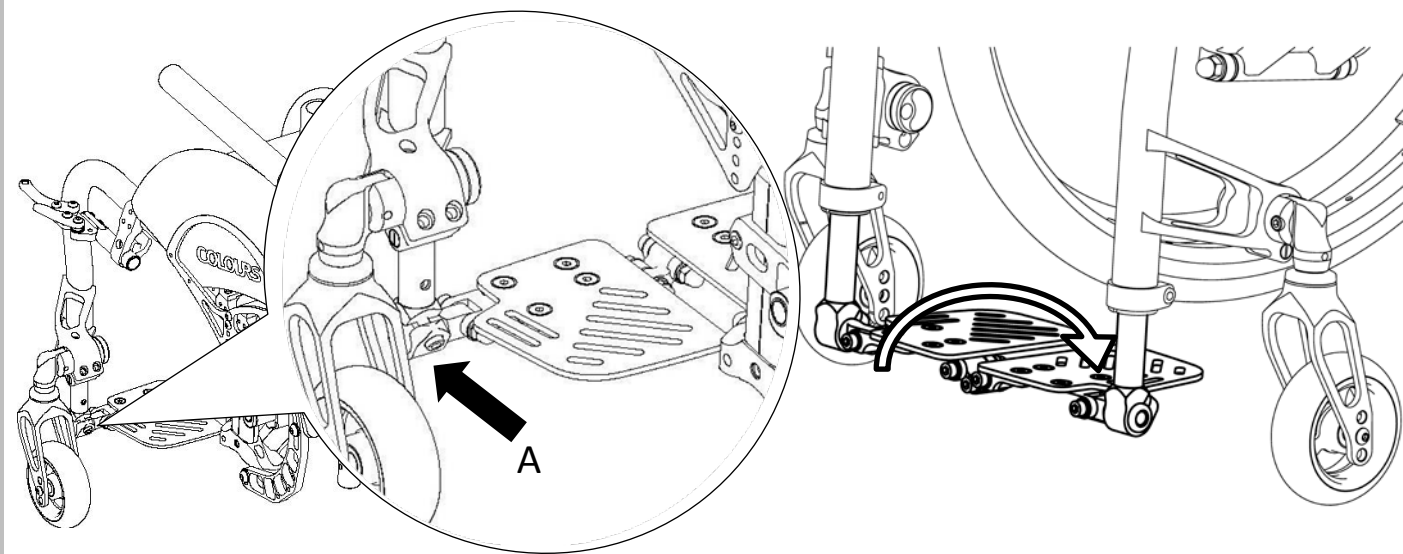
5.SET-UP & ADJUSTMENT-2

Footrest Angle Adjustment

1. Loosen set screw (A) using 5mm hex key.
2. Adjust footplate to desired angular position.
3. Tighten set screw.

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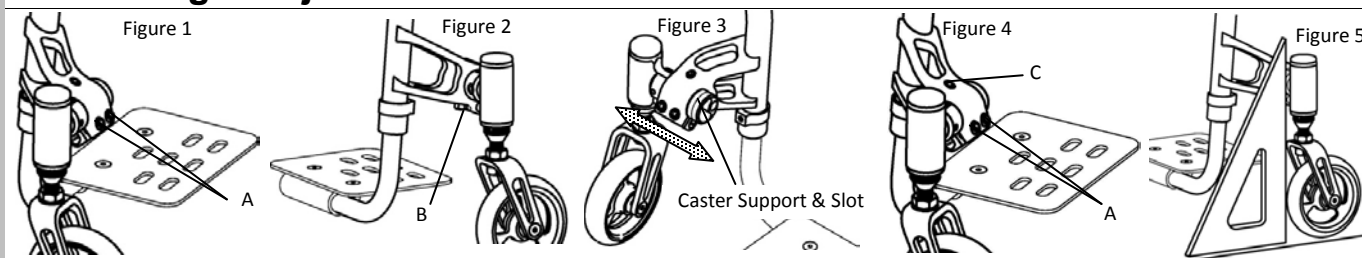
5.SET-UP & ADJUSTMENT-3

Caster Angle Adjustment - I

1. Place the chair on a level surface.
2. Loosen with 5mm hex key, but do not remove, the two Allen screws (A) that hold the caster Housing. (See Figure 1)
3. Loosen with 8mm open end wrench the locknut (B), but do not remove, while keeping the bolt from rotating. (See Figure 2)
- 4-1. Adjusting the position.
- 4-2. Move "Caster Support" to the inside or outside and check the position to see the slot. (See Figure 3)
- 4-3. Be the same position of left and right caster housing.
- 5-1. Adjusting the angle.
- 5-2. Turn the bolt(C) with 4mm hex key to adjust the angle of caster housing.(See Figure 4)
- 5-3. Adjust the caster housing to be perpendicular to the level surface.
(Use a drafting triangle or similar 90°angle tool. See Figure 5)
- 5-4. Using the Allen Wrench to prevent the bolt from turning, use the open end wrench to tighten the locknut.
- 5-5. Securely tighten the two Allen screws (A).
- 5-6. Recheck the caster housing to be certain it is still perpendicular to the ground before riding.

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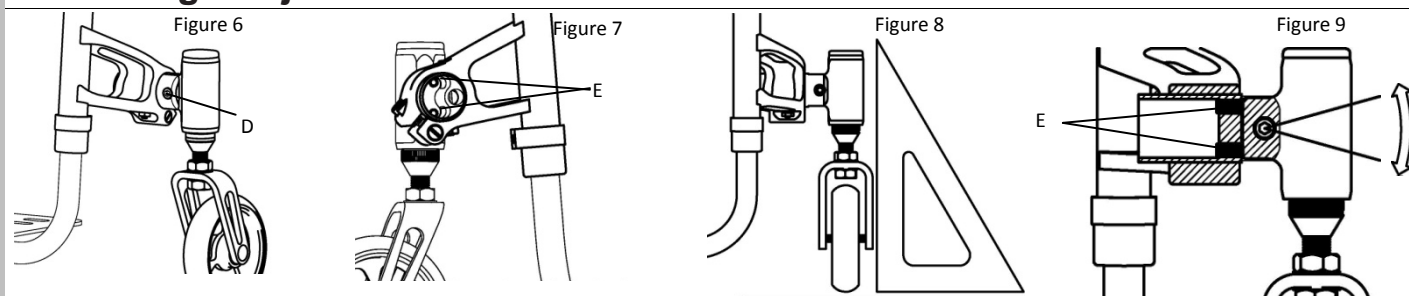
5.SET-UP & ADJUSTMENT - 4

CasterAngleAdjustment - II

- 6-1. Adjusting the camber angle of caster housing.
- 6-2. Loosen with 5mm hex key, but do not remove, the one Allen screw (D). (See Figure 6)
- 6-3. Loosen with 3mm hex key, but do not remove, the two set screws (E). (See Figure 7)
- 6-4. Adjust the caster housing to be perpendicular to the level surface.
(Use a drafting triangle or similar 90° angle tool. See Figure 8)
- 6-5. Securely tighten the one Allen screws (D).
- 6-6. Securely tighten the two set screws (E). (See Figure 9)

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
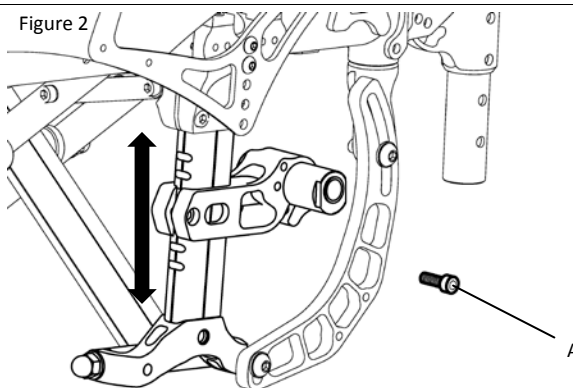
Figure 3

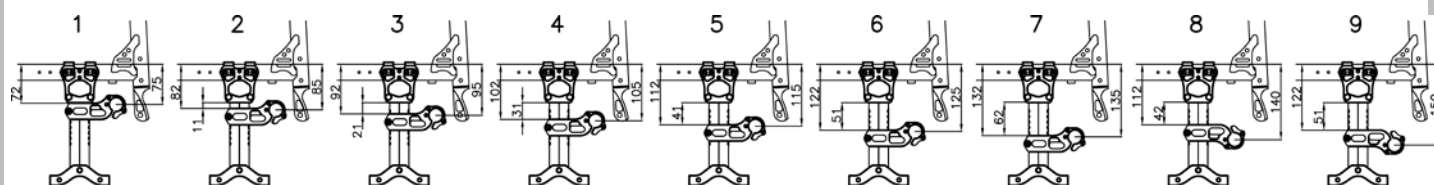1. Adjusting the Rear Seat Height.

2. Loosen with 5mm hex key, remove the one Allen screws (A). (See Figure 1,2)

3. Adjust height the axle mount (See Figure 3)

4. Securely tighten the one Allen screws (A). *A hole of the axle clamp matches a hole in the frame.

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Center of Gravity STEP1

1-1.Taking off side guards

1-2. Loosen with 4mm hex key, remove the two Allen screws (A).
(See Figure 1)

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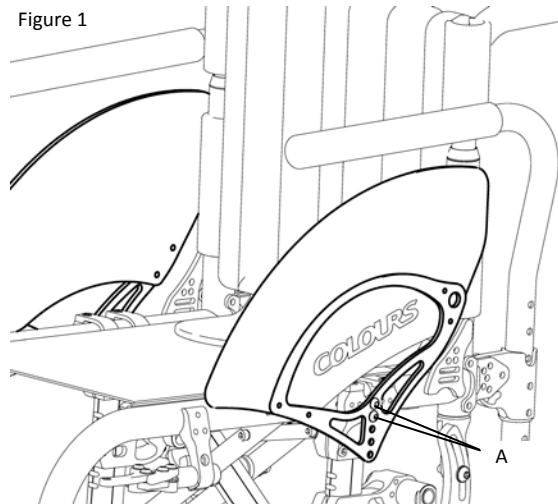
Figure 1

Center of Gravity STEP2


2-1.Loosen 4 nuts (B) with 10mm open end wrench.
But do not remove.

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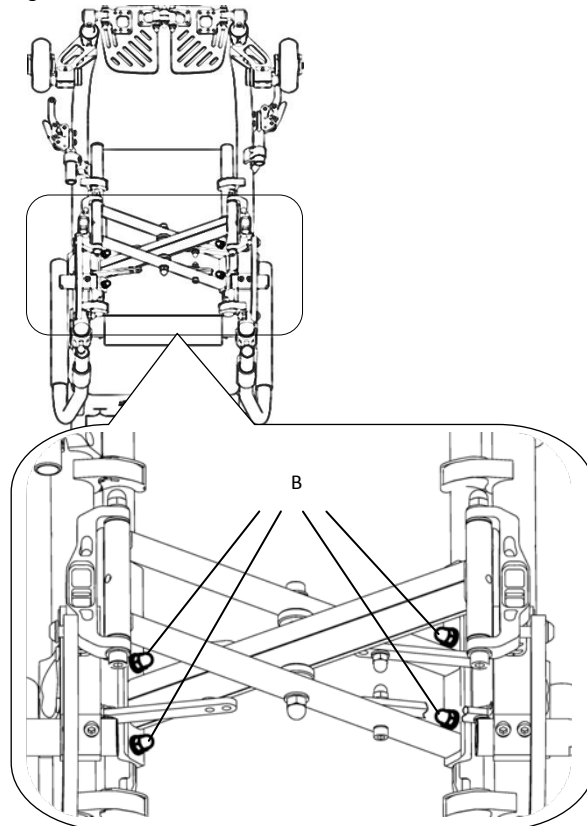
5.SET-UP & ADJUSTMENT - 7

Center of Gravity STEP3

3-1. Loosen with 5mm hex key and 13mm open end wrench the two Allen screws (C). But do not remove.(See Figure 3)

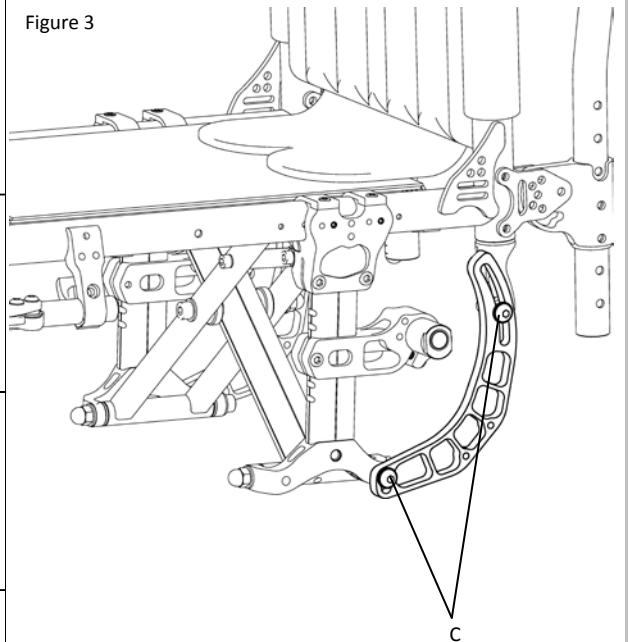Figure 3

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
Center of Gravity STEP4

4-1. Loosen with 3mm hex key, remove the two Allen set screws (D).

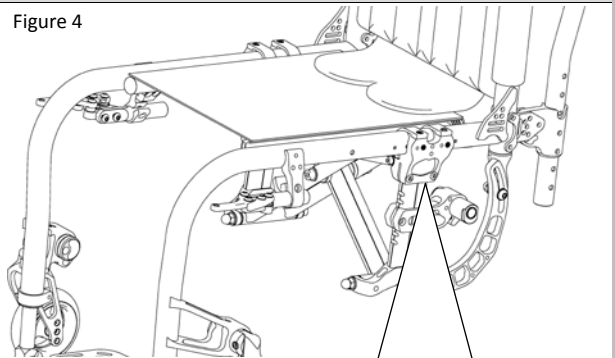Figure 4

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
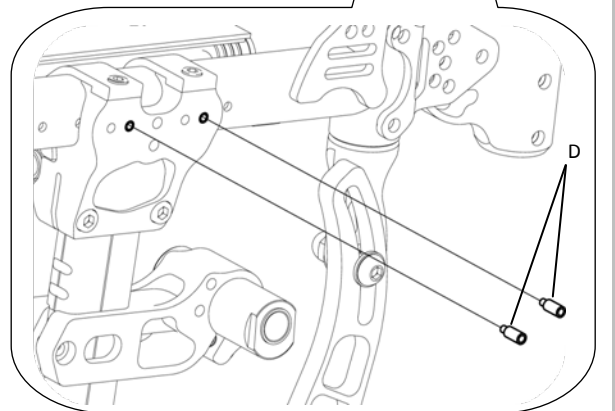
5.SET-UP & ADJUSTMENT - 8

Center of Gravity STEP5

5-1. Loosen with 5mm hex key, the two Allen screws (E). But do not remove. (See Figure 5)

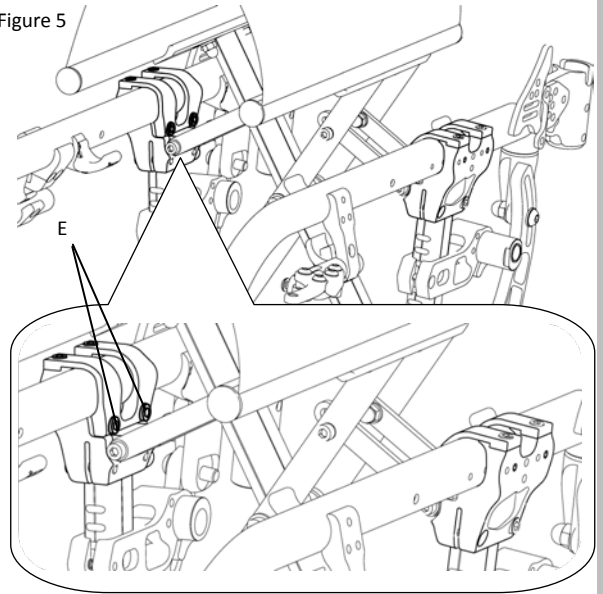Figure 5

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Center of Gravity STEP6

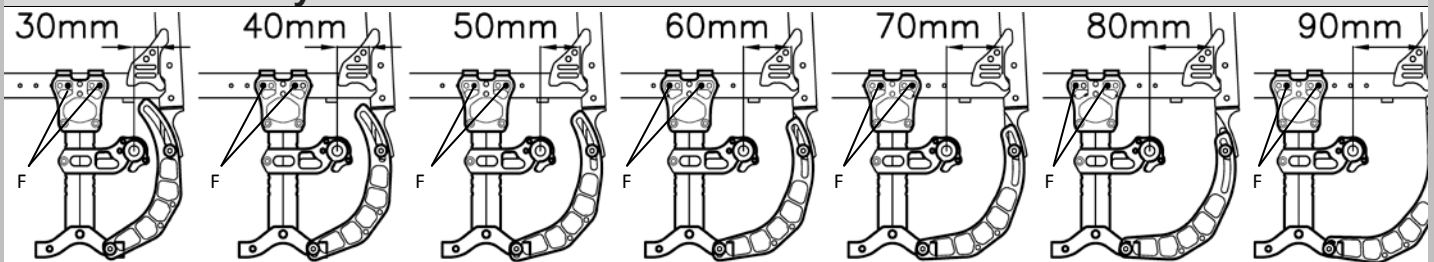
Figure 6

6-1. Slide axle clamp tube forward or back to desired position.

6-2. Axle clamp hole matches the main frame hole (F).

* Position of the matching holes varies depending on the settings for center of gravity. (See Figure 6, (F))

6-3. Insert the two Allen set screw (D) to the matching holes (F) with 3mm hex key. (See Figure 6, (F))

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5.SET-UP & ADJUSTMENT - 9

Center of Gravity STEP7

7-1. Securely tighten two Allen Screw(E). (See Figure 7)

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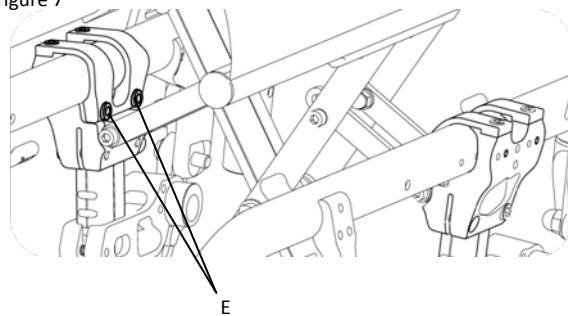
Figure 7

Center of Gravity STEP8

8-1. Securely tighten with 5mm hex key and 13mm Open End Wrench the two Allen Screw(C). (See Figure 8)

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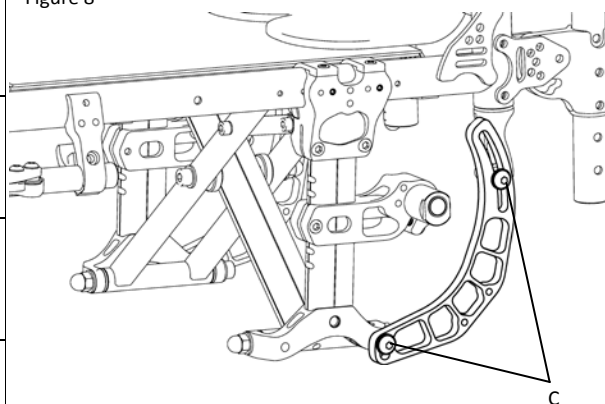
Figure 8

Center of Gravity STEP9

9-1. Securely tighten with 10mm open end wrench the four nuts (B). (See Figure 9)

*Backrest tube to seat sling must be set at 25mm(1")

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
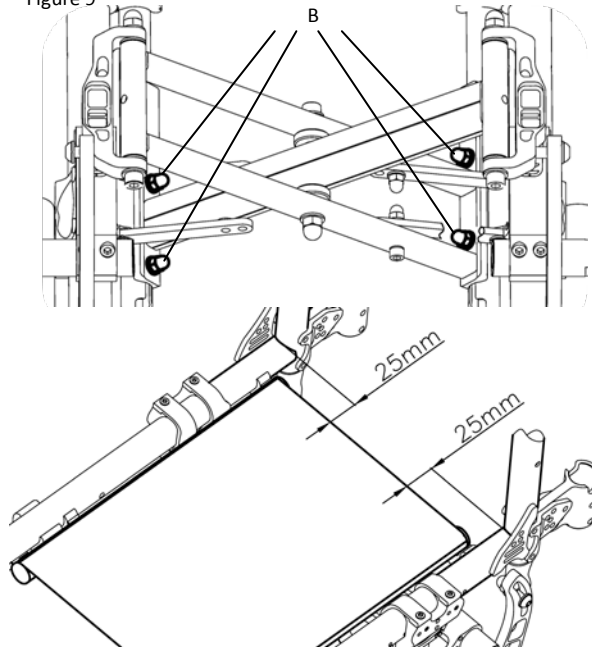
5.SET-UP & ADJUSTMENT - 10

Center of Gravity STEP10

10-1. Securely tighten two Allen screws (E). (See Figure 7)

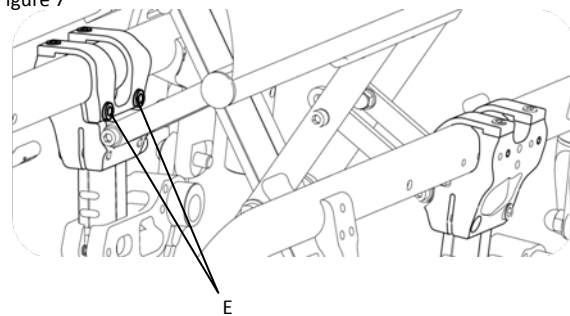Figure 7

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
Center of Gravity STEP11

11-1. Securely tighten with 5mm hex key and 13mm open end wrench the two Allen screws(C). (See Figure 8)

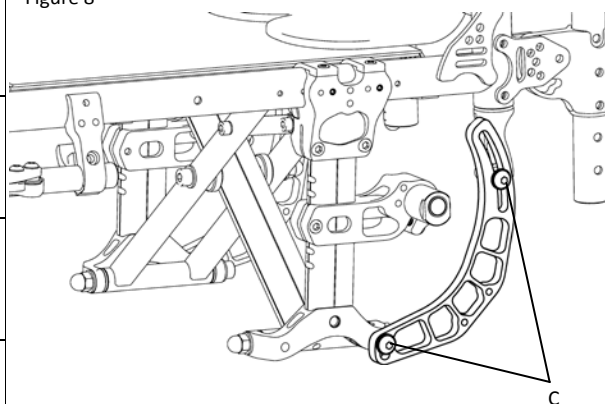Figure 8

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6. MAINTENANCE - 1

1. Proper maintenance will improve performance and extend the useful life of your chair.
2. Clean your chair regularly. This will help you find loose or worn parts and make your chair easier to use.
3. Repair or replace loose, worn, bent or damaged parts **before using this chair**.
4. To protect your investment, have all major maintenance and repair work done by your authorized supplier.

● WARNING

If you fail to heed these warnings your chair may fail and cause severe injury to the rider or others.

1. Inspect and maintain this chair strictly per maintenance chart. (Next Page)
2. If you detect a problem, make sure to service or repair the chair before use.
3. At least once a year, have a complete inspection, safety check and service of your chair made by an authorized supplier.

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6. MAINTENANCE - 2

MAINTENANCE CHART				
Weekly	3 Months	6 Months	Annually	Check
✓				Tire inflation level
✓				Wheel locks
	✓			All fasteners for wear and tightness
		✓		Armrests
		✓		Axle and axle sleeves
	✓			Quick-release axles
	✓			Wheels, tires and spokes
	✓			Casters
	✓			Anti-tip tubes
		✓		Frame and cross brace
	✓			Upholstery
			✓	Service by Authorized Supplier

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7.COLOURS WARRANTY POLICY-1

LIMITATIONS

- 1.This warranty does not apply if:
 - a. The chair is subject to abuse
 - b. The chair is not maintained as recommended in the owner's manual
 - c. The chair is transferred to a different person from the original owner
2. We do not warrant:
 - a. Tires and tubes, upholstery, pads and push-handle grips.
 - b. Damage from neglect, accident, misuse, or from improper installation or repair.
 - c. Products modified without COLOURS WHEELCHAIAR's express written consent.
 - d. Damage from exceeding the weight limit.
3. This warranty is VOID if the original chair serial number tag is removed or altered.
4. This warranty is not transferable and only applies to the first consumer purchase of this wheelchair through an authorized COLOURS WHEELCHAIAR dealer.

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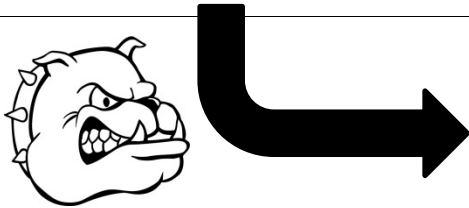
8. ENJOY THE RIDE !

Thank you for choosing a Colours wheelchair. We want to hear your questions or comments about this manual, the safety and reliability of your chair, and the service you receive from your Colours supplier. Please feel free to write or call us at the address and telephone number below:

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Tel: [951-808-9131](tel:951-808-9131) FAX: 951-808-9949

Toll Free: 800-892-8998 www.colourswheelchair.com

☞ Your authorized supplier knows your wheelchair best, and can answer most of your questions about chair safety, use and maintenance. For future reference, fill in the following:

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